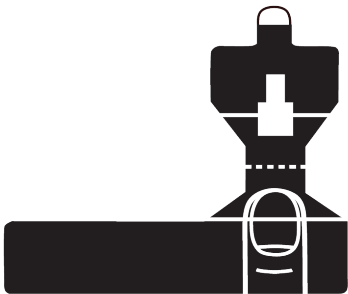
Cadwell[®] Sleepmate

Instructions for Use (IFU)

Oximetry Sensor Tape

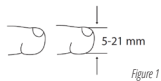
Figure 1

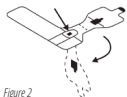
Figure 2

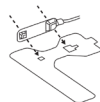
Figure 3

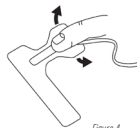
Figure 4

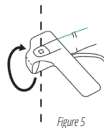
Figure 5

Figure 6

Figure 7

Contents

Page

English	4
Dansk	6
Deutsch	8
Español	10
Suomi	12
Français	14
Italiano	16
Nederlands	18
Norsk	20
Português	22
Svenska	24

Intended use

The Cadwell Sleepmate™ Sensor Tape is designed to fit flex type oximeter probes. The Cadwell Sleepmate Tape is self-adhesive, latex free and designed for single-patient use but easily reapplied. The Sleepmate Tape offers patient comfort, extended monitoring, and minimizes motion artifact. The Cadwell Sleepmate Tape is designed to fit the index finger of an adult. Other fingers (or toes) may be used if the skin thickness is between 5 and 21 millimeters. Alternative locations may give inaccurate readings as they may limit the amount of light able to pass through. *See Figure 1*

Instruction for use

Attaching the Tape to the Sensor

1. Beginning with the perforation at the top purple tab, peel the paper backing down just below the second opening of the Sleepmate Tape. *See Figure 2*
2. Firmly attach the sensor probe to the Sleepmate Tape making certain the sensor notches are lined up evenly with the Sleepmate Tape openings. *See Figure 3*
3. Peel off the remaining paper backing.

Attaching the Sensor to the Patient

1. Using the dotted line as a guide, press the patient's finger onto the adhesive side so that the top finger aligns evenly with the dotted line. *See Figure 4*
2. Fasten the bottom tabs of the tape to either side of the patient's finger.
3. Fold the tape over the tip of the finger. Check to be sure the openings in the bottom and the top of the tape are aligned evenly on the finger. *See Figure 5*
4. Fasten the remaining long tab of adhesive gently around the patient's finger. Pulling or stretching the tape around the finger may cause an irregular signal or irritate the skin. *See Figure 6*
5. The sensor cable should be fastened separately around the base of the finger. Do not fasten tightly as this may restrict blood flow. *See Figure 7*

WARNING:

Monitor patient often to ensure proper positioning of sensor, circulation and any skin irritation that could develop. Discontinue use immediately if any sign of irritation develops. Do not wrap the tape too tightly as inaccurate readings and skin injury may result.

Federal law (USA) restricts this device to sale by or on the order of a physician.

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©2022 Cadwell Industries, Inc. All rights reserved. PN 302560-639 Rev. 00 The information contained in this document is subject to change without notice. This document contains trademarks that belong to Cadwell Industries, Inc. and other companies, respectively.
Cadwell Industries Inc.
1 (800) 245-3001 U.S.A. toll-free | +1 (509) 735-6481