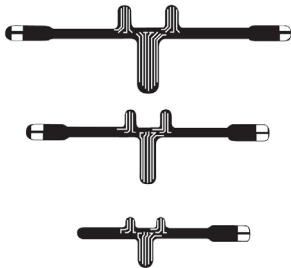
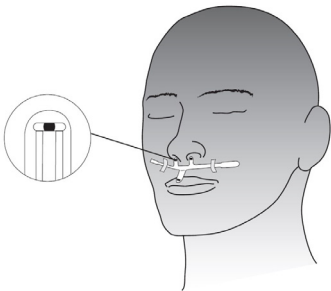
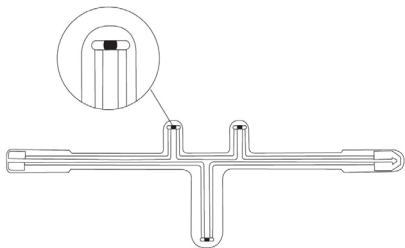
Cadwell[®] Sleepmate

Instructions for Use (IFU)

Disposable Thermistor Airflow Sensor

Contents

Page

English	4
Dansk	7
Deutsch	10
Español	13
Suomi	16
Français	19
Italiano	22
Nederlands	25
Norsk	28
Português	31
Svenska	34

Intended use

The Cadwell Sleepmate™ Disposable Airflow Sensor is intended to detect patient nasal and/or oral breathing during sleep research and clinical studies of sleep.

Instructions for use

Grasp the sensor at the exposed silver end, and insert one half of the sensor's silver end into the interface cable, then push on the neck of the sensor and insert the remainder into the cable. **Even after fully inserting the connector, a portion of the silver connector may remain exposed.** Then fold the nasal extensions at an angle away from the face to insure that they are in the flow of air. The side of the sensor with the SILVER sensor lines should be facing toward the user's skin. The one channel cable connector will point off to the patient's left.

Two Channel Output

The Two channel output allows you to see the Oral and Nasal signals separately. To use this option requires a slight modification to the sensor and the use of the two-channel cable.

Step 1: Place the sensor on a flat surface with the **BLACK** lines facing away from the patient.

Note that both ends have two silver connector plates; however, on the left end there is a green colored half circle bridging the circuit.

Step 2: With scissors or knife cut the green half circle away from the LEFT end of the sensor.

Now both ends of the sensor should look the same.

Step 3: Connect one cable to each end of the sensor.

Step 4: To obtain the **Oral signal** plug the red and black lead pair "Oral" into one channel on the recorder.

Step 5: To obtain the **Nasal signal** refer the red and black lead pair labeled "Nasal" into a second channel on the recorder. Ensure the sensor is securely attached to the patient as described and you have clear, reliable signals prior to lights out.

One Channel Output

The Adult and Medium sensors can be used as a Single channel output or a two-channel output. The Single channel output is the default setup; the Oral and Nasal airflow signals are combined on one channel. Simply plug a sensor into the single channel cable and then insert the connectors into the recording device. Ensure the sensor is securely attached to the patient as described and you have clear, reliable signals prior to lights out.

Sensor Placement

Apply tape intended for facial use to the ends of the sensor and place the sensor between the nose and mouth. Insure that the oral and nasal probes are directly in the flow of air. The sensor works best when the sensor lies flat against the skin between the nose and mouth. Run the cable over the patient's ear and apply a small piece of tape to the neck to provide a strain relief for the lead wire. Ensure the sensor is securely attached to the patient as described and you have clear, reliable signals prior to lights out.

Cleaning

This sensor was designed for a single patient use. The lead wire can be cleaned by rubbing with any cleansing agent that is not corrosive toward plastic. **The entire assembly must be completely dry before using again to insure patient safety.** For sterilization implement a standard procedure for gas sterilization. Do not autoclave this product. Use the molded ends of the connectors when removing the connector from the junction box.

Technical Specifications

Description	Properties
Sensor Sizes	Adult: 10.25 cm (4") length x 3.2 cm (1.3") (nasal tip to oral tip) x 0.03 cm (0.01") thick Medium: 8.75 cm (3.4") length x 2.75 cm (1.1") (nasal tip to oral tip) x 0.03 cm (0.01") cm thick Infant: 4.5 cm (1.8) length x 1.75 cm (0.7) (nasal tip to oral tip) x 0.03 cm (0.01") cm thick
Lead wire	Two-conductor white lead wire, approx. 200 cm (80").
Connectors	1.5 mm (0.1") safety pins
Filter Settings	Low Hz = 0.1 Hz; High Hz = 5 Hz; Gain: Output range is +/- 100 Microvolts

Caution Statement:

US Federal Law restricts this device to sale by or on the order of a physician. This product is for diagnostic purposes only and is not intended to be used as an apnea monitor or in a life supporting or sustaining capacity.

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©2021 Cadwell Industries, Inc. All rights reserved. PN 302993-639-3 Rev. 00
Cadwell Industries Inc.
1 (800) 245-3001 U.S.A. toll-free | +1 (509) 735-6481
This document contains trademarks that belong to Cadwell Industries, Inc.